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Manual
For professional use only



1,25-(OH)₂-Vitamin D₃/D₂ **IDKenhance[®] ImmuTube[®]** **LC-MS/MS extraction kit**

*For the extraction of 1,25-(OH)₂-vitamin D₃/D₂
from plasma and serum*

Valid from 2023-11-27

KM1102

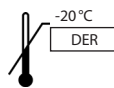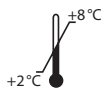Immundiagnostik AG, Stubenwald-Allee 8a, 64625 Bensheim, Germany

Tel.: +49 6251 70190-0

Fax: + 49 6251 70190-363

e.mail: info@immundiagnostik.com

www.immundiagnostik.com

Table of Contents

SAFETY INFORMATION	11
1. INTENDED PURPOSE	12
2. MATERIALS PROVIDED	12
3. MATERIALS REQUIRED BUT NOT SUPPLIED	12
4. PREPARATION AND STORAGE OF REAGENTS	13
5. SAMPLE PREPARATION	13
6. WARNINGS AND PRECAUTIONS	15
7. DISPOSAL	16
8. SYMBOL EXPLANATION	16

SAFETY INFORMATION

The assay has to be performed exclusively according to the instructions for use enclosed with the kit. Important safety information for this product can be found in chapter 6.

1. INTENDED PURPOSE

The 1,25-(OH)₂-Vitamin D₃/D₂ IDKenhance® ImmuTube® LC-MS/MS extraction kit is intended for the sample preparation of 1,25-(OH)₂-Vitamin D₃/D₂ for the 1,25-(OH)₂-Vitamin D₃/D₂ LC-MS/MS application by means of extraction from human serum and plasma via immunoaffinity enrichment (ImmuTube®) and derivatization (IDKenhance®). It is for manual use by laboratory professionals. For *in vitro* diagnostic use only.

2. MATERIALS PROVIDED

Cat. No.	Identifier	Kit components	Quantity
KM0003	WASHSOL	Wash solution	1 x 80 ml
KM1002	DER	Derivatization reagent	1 x 3 ml
KM1002	STOP	Stop solution	1 x 1.5 ml
KM1100	ELUSOL	Elution solution	1 x 15 ml
KM1100	COLUMNS	ImmuTube®-columns for extraction of 1,25-(OH) ₂ -Vitamin D ₃ /D ₂	50 Stück

For reorders of single components, please use the catalogue number followed by the identifier without space as product number.

3. MATERIALS REQUIRED BUT NOT SUPPLIED

The following material is required for the 1,25-(OH)₂-Vitamin D₃/D₂ IDKenhance® ImmuTube® LC-MS/MS extraction (not included in the kit):

- Glass tubes (inner diameter 10 mm)
- Precision pipettors and disposable tips to deliver 10–1 000 µl
- Centrifuge capable of 10 000 *g* for 1.5 ml Eppendorf reaction tubes and 550 *g* for glass tubes, respectively
- Vortex mixer
- Vacuum centrifuge or nitrogen evaporator

- Standard laboratory disposable plastic reagent vials (inner diameter 10 mm)
- Overhead rotator
- Orbital shaker

The following articles, among others, can be ordered separately for the 1,25-(OH)₂-Vitamin D₃/D₂ IDKenhance® ImmuTube® LC-MS/MS application at Immundiagnostik AG:

- 1,25-(OH)₂-Vitamin D₃/D₂ IDKenhance® ImmuTube® LC-MS/MS kit (KM1002)
- calibrators and controls (KM1002CAL1–6 and KM1002CTRL1–3)
- tuning solution for 1,25-(OH)₂-vitamin D₃/D₂ (KM1002TU)
- tuning solution for the internal standard (KM1002TS)
- UPLC column (KM1002SP)
- in-line filter (KM1002IF)
- in-line filter holder (KM1002IH)

Please ask for our single component price list.

Immundiagnostik AG offers further options for sample preparation for the 1,25-(OH)₂-vitamin D₃/D₂ LC-MS/MS application. Please contact us for your individual solution.

4. PREPARATION AND STORAGE OF REAGENTS

The test reagents should be stored protected from light, dry and their specified storage temperature (DER: -20 °C; all others: 2–8 °C). The test reagents stored in this way are usable until the indicated expiry date.

5. SAMPLE PREPARATION

Serum and plasma samples are suitable for the assay.

The samples must be centrifuged before use (minimum 5 min at 10 000 *g*).

Control samples should be carried along with each run.

Prior to use in the assay, allow all samples and reagents to come to room temperature (18–26 °C).

Mix samples and reagents well before use.

1.	Carefully vortex ImmuTubes® and centrifuge (30 s at 500–1 000 rpm) to ensure that no suspension remains in the lid.
2.	Label the ImmuTubes®, open ImmuTubes®, quickly add 500 µl sample, close ImmuTubes® and mix gently.
3.	Incubation for 1 h at room temperature in an overhead rotator (15–30 rpm).
4.	Insert closed ImmuTubes® in plastic reagent vials and centrifuge for 2 min at 550 g to ensure that no suspension remains in the lid.
5.	Open the outlet of the ImmuTubes®, then the lid and centrifuge for 2 min at 550 g to dryness. Discard flow-through.
6.	Add 500 µl WASHSOL and centrifuge for 2 min at 550 g to dryness; discard the flow-through. Perform this washing step a total of three times.
7.	Label new glass tubes and place ImmuTubes® in the labelled glass tubes.
8.	Add 250 µl ELUSOL, centrifuge for 2 min at 550 g and collect the eluate with the analytes in the glass tubes.
9.	Evaporate the solvents under a nitrogen stream at 37 °C or in a vacuum centrifuge.
10.	Add 50 µl DER into each glass tube and incubate for 1 h at 1 200 rpm on a shaker in the dark (attention: reaction is light sensitive).
11.	Add 25 µl STOP into each glass tube and incubate for 2 min at 1 200 rpm on a shaker.
12.	Evaporate the eluate under a nitrogen stream at 37 °C or in a vacuum centrifuge.
13.	Dissolve the residue in 175 µl solvent (adjust solvent to mobile phase).


6. WARNINGS AND PRECAUTIONS

- This product has been produced and placed on the market in accordance with the regulation (EU) 2017/746 (IVDR).
- IDKenhance® and ImmuTube® are brands of Immundiagnostik AG.
- All reagents in the kit package are for *in vitro* diagnostic use only.
- The assay should always be performed according the enclosed manual. Incubation time, incubation temperature and pipetting volumes of the components are defined by the producer. Any variation of the test procedure, which is not coordinated with the producer, may influence the results of the test. Immundiagnostik AG can therefore not be held responsible for any damage resulting from wrong use.
- Do not mix different lot numbers of any kit component. Plugs and caps of different reagents should not be swapped.
- Reagents should not be used beyond the expiry date stated on the kit label.
- Serious incidents are to be reported to Immundiagnostik AG and the national regulatory authorities.
- Please contact Immundiagnostik AG if one or more components of the kit are damaged, missing (see material supplied) or precipitates are visible in the ready-to-use solutions.
- Warranty claims and complaints regarding deficiencies must be logged within 14 days after receipt of the product. The product should be send to Immundiagnostik AG along with a written complaint.
- The GHS symbols indicated on the individual components and specifications of the material safety data sheets (available on request from Immundiagnostik AG) must be noted. When working with these reagents, the legal protective precautions must be adhered to.

7. DISPOSAL

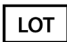Derivatization reagent (DER), stop solution (STOP) and elution solution (ELUSOL) must be disposed as non-halogenated solvents.

8. SYMBOL EXPLANATION

	Temperature limitation		Catalogue number
	In Vitro Diagnostic Medical Device		To be used with
	Manufacturer		Contains sufficient for <n> tests
	Lot number		Use by
	Do not re-use		Consult instructions for use
	Consult specification data sheet		

Immundiagnostik AG

Stubenwald-Allee 8a
64625 Bensheim, Germany

Tel.: +49 6251 70190-0

Fax: +49 6251 70190-363

info@immundiagnostik.com

www.immundiagnostik.com